

CareBidets

Installation & User Manual
Flexible Armrest
Model TGH



CE



Read this manual
before use

Important safety guards	3
Intended use	3
Contra-indication for use	3
Warranty.....	3
Included in the box.....	3
Technical specifications	5
Cleaning, disinfection & maintenance.....	6
Operation of OneTouch button (Armrest OT).....	6
<i>How it works</i>	6
Instruction for installation	7

Distributor:

J. Honoré Care ApS
Karetmagervej 19B - 7000 Fredericia
Phone: 75 56 32 44
info@honorecare.dk

Manufacturer:

Santis Kft.
Jégvirág utca 18
4034 Debrecen
Hungary



Important safety guards



Do not step on the product	Warning – Danger of falling
Do not stand on the product	Warning – Danger of falling
Do not be used if not properly mounted	Warning – Danger of falling
Do not use if product is broken	Warning – Danger of personal injury
Do not use cigarettes and other inflammable materials in close proximity to the product	Warning – Danger of personal injury
Transportation & Storage	The product should always be stored and transported under dry conditions

Intended use

CareBidets flexible armrest may only be installed on toilet with a normal toilet seat or in combination with an electronic bidet from CareBidets. The product serves the purpose of supporting the user getting on and of the toilet. The armrest can also work in combination with a toilet seat raiser from CareBidets with product code 600,601,650,651. The product conforms to the requirements of the Medical Device Regulation (EU) 2017/745. In case of an adverse event occurred in relation to use of the device, it should be reported to your local dealer or the manufacturer and the National competent authority in a timely matter.

Contra-indication for use

- The patient should weigh less than 150 kg.
- The patient/user is not physically and mentally able to operate the product safely

Warranty

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries. Warranty claims can only be made through the provider from whom the appliance was obtained. Please contact your local sales agent.

Included in the box

2 x Arm handles
1 x Main module
2 x White nut cover
2 x Long bolts incl. fittings <i>(If ordered together with the raiser from CareBidets)</i>

Technical specifications

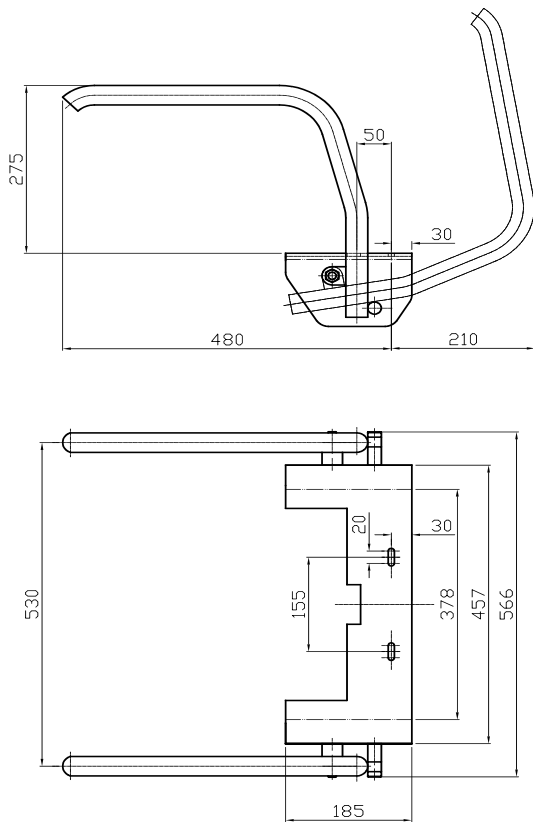


Figure 1: Technical Drawing (mm.) - Flexible Armrest

Description	Data
Max width	566 mm
Width between arm handles	485 mm
Hight – from seat to upper armrest	255 mm
Max user weight	150 kg.
Fits on following CareBidets models	Jasmin Care II (JHC611/612), Jasmin Care (JHC511/512), Lotus Care (JHC311/312)
Models	<ul style="list-style-type: none"> • Armrest standard. <ul style="list-style-type: none"> ○ Product code.: 500 • Armrest with inbuild OneTouch control <ul style="list-style-type: none"> ○ Product code: 5000T
CE	MDR 2017/745

Cleaning, disinfection & maintenance

When cleaning the product do only use mild detergents. Don't use highly corrosive acidic or alkaline products as it may damage the surface. Wipe all parts with a damp cloth with some mild detergents and finish with a dry cloth.

The product can be cleaned and disinfected using commercially available agents. Disinfect the product by wiping down all generally accessible surfaces with disinfectant.



The product must be checked regularly for damage and to ensure that it is securely assembled and fastened. Call for technical assistance if the product is damaged or not properly fastened.

Operation of OneTouch button (Armrest OT)

This section is only relevant for Armrest OT (Item no. 5000T)

The inbuilt button at the end of the armrest can be used to operate the functions of CareBidets Jasmin Care / Lotus Care (electronic bidet seat). Press the button once and the electronic bidet will operate your personal and automatic cleaning and drying program. See the user manual of your CareBidets of how to install the automatic functions.

How it works

- Pull out the wire at the bottom of the armrest – Connect it to the AUX port of the remote control

When seated

- Press one time = Auto 1 program starts – Press again if you wish to stop the program a head of time.

When NOT seated

- Press twice = Orientation light (Blue LED-light) turn on / off



Figure 2: Armrest with OneTouch operation of CareBidets

Instruction for installation


- 1) Remove the plastic mounting plate from the Stainless-steel plate. Save it for later use.**




- 2) Install bolts, screws and plastic plate on the armrest module**



3) Bolts are first placed in the prepared holes on the armrest module.

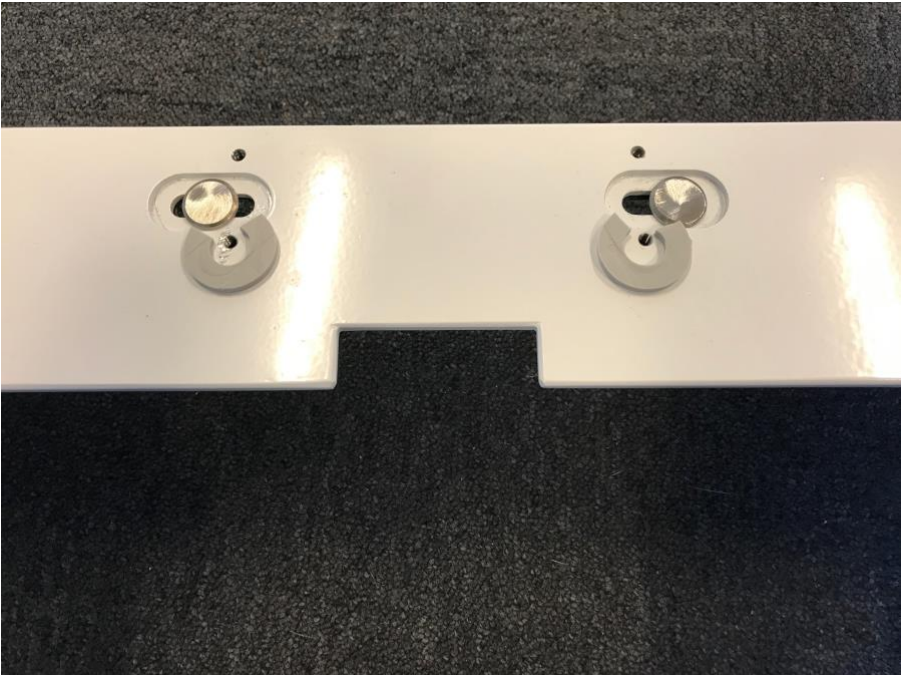

4) Attach the plasticplate on the module.

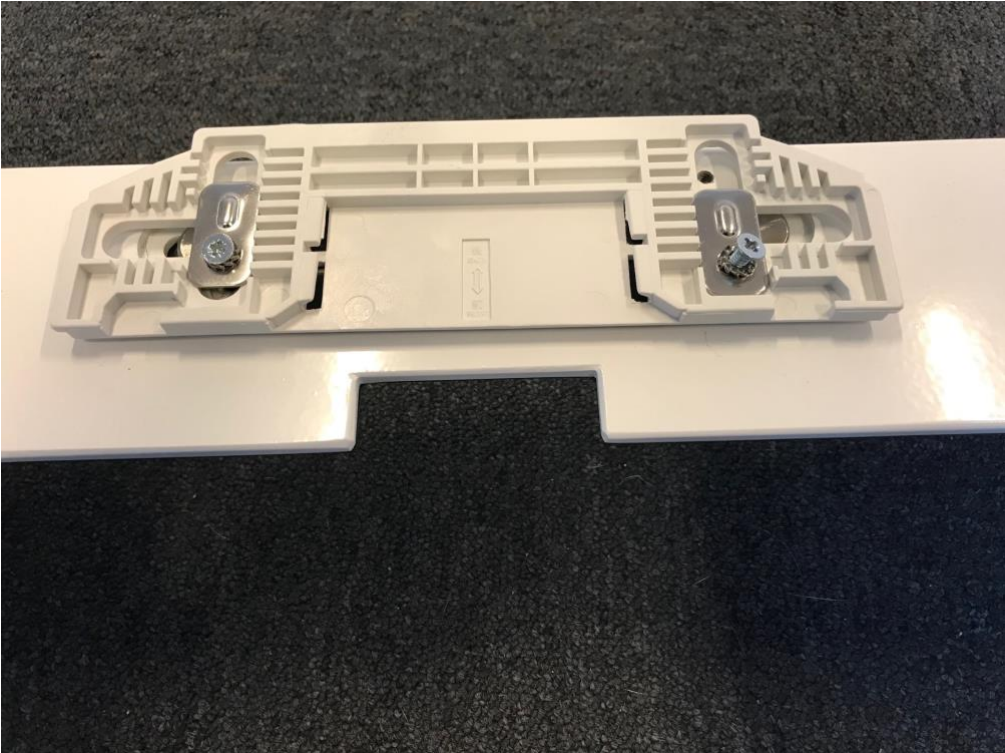

5) Remove nut and spring washer



6) Leave the washer on the module



7) Install armrest using spring washer and nut



8) White nut cover



- 5) Place the assembled armrest module over the toilet. Install and fasten it to the toilet bowl with the fitting supplied together with the mounting plate.

Adjust the plastic mounting plate and fasten it in the correct position.

CareBidets can now be installed. Now follow the instructions supplied with the CareBidets.

